



activheal

WOUND CARE
BECAUSE YOU CARE



HIGHLY
ABSORBENT



HIGH WET
STRENGTH



REDUCES RISK
OF MACERATION



INCREASED
VERTICAL WICKING

AQUAFIBER EXTRA

WOUND CARE
BECAUSE YOU CARE

FEATURES

Highly absorbent // High gelling // The dressing can remain *in situ* for up to 7 days // Reduces risk of maceration // Maintains a moist wound healing environment // Aids autolytic debridement // High wet tensile strength //

ACTIVHEAL® AQUAFIBER EXTRA CREATES A MOIST WOUND HEALING ENVIRONMENT FOR MODERATE TO HEAVILY EXUDING WOUNDS.


ActivHeal® Aquafiber Extra is an innovative combination of two proven wound care technologies based on CMC & alginate fibres. A hidden integrity layer in the dressing supports the tensile performance.

ActivHeal® Aquafiber Extra is a soft, conformable, highly absorbent and non-woven dressing which turns into a soft, cohesive gel when in contact with wound exudate. Thus it helps with autolytic debridement and maintains a moist wound healing environment following the principles of advanced wound care. ActivHeal® Aquafiber Extra effectively addresses the key challenges of improved exudate management, gelling & tensile performance.

PERFORMANCE


High absorbency

The unique design & fibre engineering of ActivHeal® Aquafiber Extra allows the dressing to absorb exudate & transform into a cohesive gel.

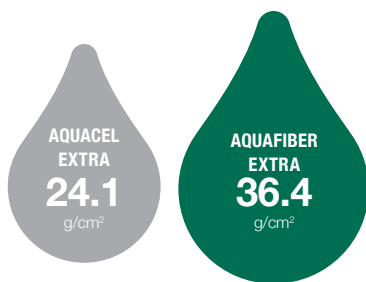

High wet tensile strength


The quilting pattern of ActivHeal® Aquafiber Extra is supported by a hidden integrity layer allows for intact removal of the dressing.

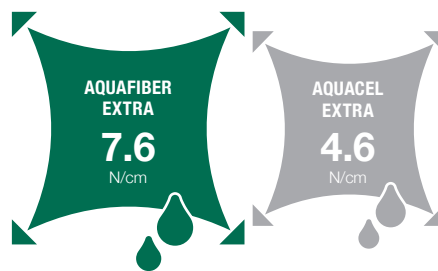

Low lateral wicking

In-house analysis¹ has confirmed that the vertical wicking of the ActivHeal® Aquafiber Extra can be potentially helpful in the prevention of maceration within the peri-wound skin.

- ✓ HIGH ABSORBENCY¹
- ✓ HIGH WET TENSILE STRENGTH¹
- ✓ INCREASED VERTICAL WICKING¹

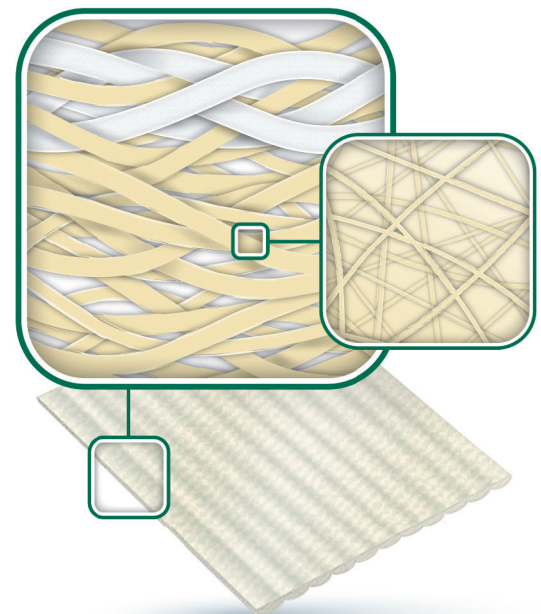

INDICATIONS

ActivHeal® Aquafiber Extra is indicated for the management of the following moderately to heavily exuding acute and chronic wounds:

- Pressure ulcers
- Venous leg ulcers
- Arterial leg ulcers
- Diabetic ulcers
- Graft and donor sites
- Post-operative surgical wounds
- Cavity wounds
- Superficial and partial thickness burns

Moderate > Heavy Exudate



Wound Types



SIZES AND CODES

ACTIVHEAL® AQUAFIBER EXTRA

SIZE (CM)	QTY PER CARTON	QTY PER CASE	PRODUCT CODE	NHS SUPPLY CHAIN CODE	DT CODE
5x5	10	10	9023625	ELY795	407-5339
10x10	10	10	9023632	ELY796	407-5347
15x15	5	10	9023649	ELY797	407-5628
20x30	5	10	9023656	ELY798	407-5321
2.5x30.5 Ribbon	5	10	9023663	ELY799	407-5602
2x46 Ribbon	5	10	9023670	ELY800	407-5610

ActivHeal® Aquafiber Extra is available through the NHS Supply Chain and Drug Tariff

Advanced Medical Solutions Ltd

Premier Park, 33 Road One, Winsford Industrial Estate, Winsford, Cheshire CW7 3RT
 T 08444 125 755 E activheal@admedsol.com W www.activheal.com



References 1. AMS Data on file LD024-16, LD037-16.

ActivHeal is a Registered Trademark of Advanced Medical Solutions. Aquacel Extra is a registered trademark of ConvaTec.

© Advanced Medical Solutions 2020 // Review date May 2020 // Document code AMS/AH-012-01